



# HD SUPPLIES

44 Rabans Close  
Rabans Lane Ind. Estate  
Aylesbury  
Bucks HP19 8RS, UK  
Telephone: 01296-431920  
Fax: 01296-392121  
E-mail: [webenquiries@hdsupplies.net](mailto:webenquiries@hdsupplies.net)

## CF PRODUCT INFORMATION

### RICHARDSONS PRESERVED GUINEA PIG COMPLEMENT – GP2, GP5

#### Package

2ml or 5ml Lyophilised


#### Stability and Storage

Store lyophilised product at 4°C. Shelf life is a minimum of two years. Reconstitute with 2ml or 5ml distilled water. Undiluted complement will keep for at least a week at 4°C.

#### Instructions for Use

1. Add 2ml or 5ml of sterile distilled water and agitate gently for 5 minutes to reconstitute lyophilised product.
2. Add one part of product (1 above) to 7 parts of sterile distilled water. This dilution is equivalent to a 1/10 dilution of the original guinea-pig serum.
3. All subsequent dilutions should be made in Barbitone Buffered Saline (Veronal Buffer)\* according to a previously determined level and should only be used on the day of dilution and then discarded.

*\* Available from HD Supplies. All other reagents required for Complement Fixation Tests are available - please enquire.*

 European conformity according to the "IN-VITRO MEDICAL DEVICES DIRECTIVE 98/978/EC, ANNEX III". Manufactured by HD Supplies.

GP2/5/11/2003